



Your Nebuliser Care Card

To purchase your replacement nebulisers and accessories, or for more information, please call our UK Customer Services Team (Mon-Fri, 9am-5pm)

0800 1300 845

Or purchase Online 24/7*
www.respironicsonline.co.uk

*Online service only available for UK Private Patients

PHILIPS

RESPIRONICS

How to clean your **reusable** Sidestream nebuliser

Your nebuliser is for your use only and should not be shared.

Keep your nebuliser dry and clean to prevent germs from growing.

1. After **every** use, disconnect the nebuliser from the tubing and briefly turn on the compressor to clear any moisture from the tubing.
2. Remove the tubing and mouthpiece/mask from the nebuliser.
3. Unscrew the nebuliser into the three separate parts.
4. Wash the nebuliser parts and mouthpiece/mask (NOT the tubing) in warm soapy water, and then rinse them in clean and cold water.
5. Allow the parts to dry naturally and completely.

Once a week you must boil your reusable NEBULISER ONLY in water for 6-10 minutes, with 2-3 drops of washing-up liquid. This unblocks the jet holes of medication. Disassemble the nebuliser before boiling.

CAUTION: Do not use a brush/paperclip/needle to clear the jet holes.

Replacing your **reusable** nebuliser and accessories

To ensure that you are receiving the most from your treatment we recommend that you replace your reusable Sidestream nebuliser, tubing and mouthpiece/mask on a **yearly basis**.

In addition, the compressor's air inlet filter should be replaced every **3 months**, or when it no longer looks white and clean.

I need to replace my **nebuliser** and **accessories** in _____ (month)

I need to replace the **filter** in ____ / ____ / ____ / ____ (months)



Reusable Sidestream Nebuliser and Accessories - Light Blue